

Instrumentarium Imaging Announces Panoramic Radiography Modifications (3/04)

Instrumentarium Imaging has introduced a modification for the OP100, OC100 and OrthoID radiographic units. The design change will allow film cassette insertion into units only in the correct orientation and this change is intended to prevent inadvertent backward placement of the cassette into the unit that will result in a reverse-image film. The revision involves the addition of asymmetric pins to the top of the 15 x 30 cm film cassette and custom cassette holder guides that necessitates insertion in only one orientation.

New Cassette Design with Pins




Insertion of New Cassette into Modified Tunnel



Existing units can be upgraded with the purchase of kit that contains both the new tunnel end guides and cassettes.

Upgrade Kit

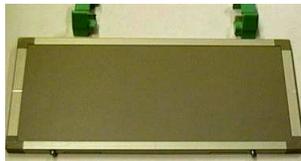

Upgrade Kit Type	Kit code
Cassette with Kodak Lanex Medium screens with guides	60761
Cassette with Kodak Lanex Regular screens with guides	60762
Cassette with Kodak Ektavision screens with guides	60763
Cassette without screens with guides	60764

Existing cassettes can be used but will require shipment to the manufacturer for the proper placement of guide pins. To date, the manufacturer has not provided government prices or the time involved for factory upgrade. For pricing and further information please contact Instrumentarium Imaging (800) 558-6120, (414) 747-1030, (414) 481-8665 FAX or info@usa.instrumentarium.com