

Propaq LT Monitors / Welch Allyn Inc.-URGENT MEDICAL ADVICE CORRECTION (1/14)

Reason/Information:

Welch Allyn issued an Urgent Medical Device Correction on the following materiel (See Image on Message under Additional Documentation/Attachment). Reason: Welch Allyn states that the monitors may exhibit hardware or software failure or electrical noise potentially caused by proximal defibrillation activity.

Disposition/Instructions:

Welch Allyn Inc. recommends that you restart your device by pressing the power (on/off) button to bring it back to full operational capacity. Welch Allyn states that it is developing a software upgrade to address the above problem. Upon release, the software update will be available via a download from Welch Allyn's website at www.welchallyn.com/propaqitupgrade.

The software update will also be included if you schedule routine or other services with Welch Allyn or an authorized service provider.

General questions, call 800-535-6663 or 315-685-4100.

Item Information:

NSN (FSC-NIIN): 6515-015469366
Nomenclature: PROPAQ LT MONITORS WITH NELLCOR SPO2
UI: EA
Manufacturer: WELCH ALLYN INC.

LOT / SN:
KA002129_THRU_KA027191
KA050687 KA124186 KA21398
KA21399 KA21407 KA21591 KA25981

NOTES:

NOTE 1:
Model #802LTAN, 802LTON, and 802LTRN
NSN (FSC-NIIN): 6515-NS1
Nomenclature: PROPAQ LT PATIENT MONITORS WITH NELLCOR SPO2
UI: EA
Manufacturer: WELCH ALLYN INC.

LOT / SN:
KA002129_THRU_KA027191
KA050687 KA124186 KA21398
KA21399 KA21407 KA21591 KA25981

NOTES:

NOTE 1:

Model #802LTAN, 802LTON, and 802LTRN

Service/Additional Instructions:

CONSIGNEES:

Wright Patterson AFB

Hulburt Field

Travis AFB

Kirtland AFB

MCGuire AFB

Holloman AFB

Ramstein AB

Aviano AB

USNH Pensacola

USNH Guantanamo Bay

USNH Camp Pendleton

Fort Belvoir COMM HOSP

Winn ACH Fort Stewart

BAMC Fort Sam Houston

Raymond Bliss ACH Fort Huachuca

USAMMA MAINT OP

Bagram Army MLC

Tripler AMC

US ARMY SPECIAL OPERATIONS COMMAND REDISTRO CTR

ARMY:

Army activities in possession of subject equipment are required to report the status of their work to the USAMMA NMP NLT:06-FEB-2014.

For Questions or Comments, contact: USAMMA NMP

Email: usarmy.detrick.medcom-usamma.mbx.nmp-mmqc@mail.mil

Phone (COMM): 301-619-6648/4464/3170

Phone (DSN): 343-6648/4464/3170

POC Contact Information:

USAMMC-E

Email: USAMMCE_QA@AMEDD.ARMY.MIL

Phone (COMM): 011-49-6331-86-7118/7181

Phone (DSN): 495-7118

Fax: 6218

US ARMY

Email: usarmy.detrick.medcom-usamma.mbx.doc@mail.mil

Phone (COMM): 301-619-4300

Phone (DSN): 343-4300

Fax: 301-619-4468

US AIR FORCE

Email: afmoa.sgalc@us.af.mil

Phone (COMM): 301-619-4170
Phone (DSN): 343-4170
Fax: 301-619-2557
US NAVY
Email: MMQC2@MED.NAVY.MIL
Phone (COMM): 301-619-3085
Phone (DSN): 343-3085
Fax: 301-619-2473

Additional Message Recipients:

Please ensure widest dissemination of information to all interested entities.

Message Dissemination Authorization:

US AIR FORCE

AF Activities Will Take Action As Prescribed In AFI 41-209, Medical Logistics Support, Chapters 3 And 9. For MAJCOMS & NGB--This Msg Has Been Transmitted To All Designated Subordinate Medical Activities.

US ARMY

See Army Regulation (Ar) 40-61, 28 January 2005, Chapter 4, And The Department Of The Army Supply Bulletin (Sb 8-75-11) For Applicable Policies And Procedures.

Additional Documentation/Attachments:

http://www.usamma.amedd.army.mil/assets/apps/nala_qaweb/attachments/MMQC-14-1008_1.pdf